



NEOGEN Vet

AluSpray[®]
Aerosol Bandage

Spray it on and it stays!

The Water Resistant Aerosol Bandage

Water Resistant

AluSpray contains a proprietary carrier that causes the active wound-protective formula to adhere to the skin without washing off in hot or cold water. AluSpray can be easily removed with soapy water to inspect the wound. The usual reapplication of AluSpray is daily to every three days to insure that an optimal bacterial barrier is maintained.

Essential To Wound Recovery

AluSpray Aerosol Bandage allows the wound to oxygenate (breathe), aiding wound healing. AluSpray may also be applied over ointment or salves.

First Aid

AluSpray is a must have product for equine and farm first aid kits due to frequent injuries and accidents experienced by horses and farm animals.

Safe

AluSpray may be used safely on superficial wounds for protection against bacterial contamination. AluSpray does not contain sulfa or other antibiotics, making it permissible to use on food animals.

Small Animals

AluSpray can be used for wound treatment in dogs and cats as well.

Features

- No antibiotic residues
- Fast and easy spray application
- Allows the wound to breathe
- Reduces risk of infection
- Protection without pain
- Exceptional adhesion
- Removable with soap
- Essential to all first aid kits
- Useful in both large and small animals

Great for:

- Dehorning
- Castrating
- Tail docking
- Summer sores
- Minor cuts and scrapes





AluSpray[®]

Aerosol Bandage



Description

AluSpray is a convenient, water-resistant aerosol bandage that creates a protective barrier against external irritants in simple wounds of small and large animals.

AluSpray is ideal for application to minor cuts and abrasions.

Item No.	Description
08887	AluSpray 75g Can – 12/Case

ORM-D (Must be shipped by ground only)

Each gram contains:

Aluminum Powder40 mg

Directions for Use

Shake well before use. Clean the area to be sprayed. Apply a thin layer of AluSpray powder to the area. Repeat daily or as directed by a veterinarian.

Warnings

- External use only.
- For veterinary use only.
- Keep out of reach of children.
- Do not spray in or near the eyes.
- Not intended for use in deep, infected, open/drainage or complex wounds. If in doubt, consult a veterinarian.

Caution

Contents under pressure. Do not use in the presence of an open flame or spark. This product stains. Avoid contact with hands, clothes, and objects.

Storage

Do not place in hot water or near radiators, stoves or other sources of heat. Do not puncture or incinerate container. Do not store at temperatures above 50°C (122°F).



944 Nandino Blvd. Lexington, KY 40511
 800-621-8829 (USA/Canada) or 859-254-1224
 info@neogen.com | animalsafety.neogen.com

