

KINETIC TECHNOLOGIES LLC.
961 BEASLEY STREET, SUITE 270, LEXINGTON, KY, 40509

Telephone: 859-258-9160

Toll-Free: 877-786-9882

Fax: 859-258-9188

Website: www.kinetictech.net

Email: customerservice@kinetictech.net



Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

KINETIC VET™ ICHON™ (STERILE POLYSULFATED GLYCOSAMINOGLYCAN)



Kinetic

Sterile Post-Surgical Lavage

For veterinary use only.

Caution: Federal law restricts this device to sale by or on the order of a licensed veterinarian.

Indications for Use:

- Ichon is an intra-peritoneal prophylactic agent for prevention of adhesions.
- Post-surgical lavage to prevent adhesions in gynaecological procedures.
- Strengthen post-surgical scar.
- For use on cats, dogs and horses.

Directions for Use:

- Install this medical device as the final lavage prior to surgical closure of the abdominal incision.
- Install at a rate of 0.013 grams / kg in a 0.9 % sterile NaCl solution.
- Volume to be determined by veterinary surgeon.
- Discard any unused portion of vial contents.

Warning: Not for food animal use. Keep out of reach of children.

Storage: Store in a cool, dry place. Do not freeze.

Manufactured for: **KINETIC™**, P.O. Box 12388, Lexington, KY 40583

877-786-9882

Fax: 859-258-9177

www.KineticVet.com

Net Contents:	
5 vials (5mL each/100mg per mL)	8/09

CPN: 6128036.0