

H.W. NAYLOR CO. INC.
121 MAIN ST., P.O. BOX 190, MORRIS, NY, 13808-0190

Telephone: 607-217-0011
Fax: 866-819-5812
Website: www.drnyaylor.com
Email: info@drnyaylor.com

Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

DR. NAYLOR[®] TEAT DILATORS

H.W. Naylor

Active Ingredient(s): DR. NAYLOR[®] Soft Surface Dilators are packed in a special lubricant containing petrolatum, water, lanolin, beeswax, paraffin, isopropyl alcohol 1.2%, methylparaben, propylparaben, sodium borate, oil of clove and color.

Indications: DR. NAYLOR[®] Dilators keep the teat open in the treatment of cut, bruised, and sore teats.

Assists in preventing scar tissue obstructions while the teat heals.

Dosage and Administration:

1. Thoroughly cleanse the end of the teat with soap and water. Disinfect with a mild antiseptic or dairy disinfectant solution. Insert the DR. NAYLOR[®] Dilator in the teat and allow it to remain there until the next milking.
2. At the next milking, remove the dilator and discard it. Discard the first few streams of milk in a strip cup, then milk out the quarter. Clean and disinfect the end of the teat each time before inserting the dilator, and use a new DR. NAYLOR[®] Dilator for each insertion.
3. Keep a DR. NAYLOR[®] Dilator in the teat between milkings until the teat milks freely by hand.

Precaution(s): Store in a clean, dry, cool place.

Caution(s): Keep out of the reach of children.

Thoroughly clean the udder and teats before milking. Do not remove the cylinder from the box. Keep the lid on.

Presentation: Package of 40 dilators.

CPN: 1089007.0

By using this content, you agree to the [Terms of Service](#) and [Privacy Policy](#).

Copyright 2020 - Animalytix LLC