

**INNOVACYN, INC.**  
**3546 N. RIVERSIDE AVE., RIALTO, CA, 92377**

Toll-Free: 866-318-3116  
Fax: 909-428-1947  
Website: [www.innovacyn.com](http://www.innovacyn.com)  
Website: [www.vetericyn.com](http://www.vetericyn.com)  
Email: [customerservice@innovacyn.com](mailto:customerservice@innovacyn.com)

---

Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

---

## **VETERICYN PLUS<sup>®</sup> VF ANTIMICROBIAL WOUND BARRIER HYDROGEL**

**Innovacyn**

**HIGH-VISCOSITY HYDROGEL**

Wound Protection

Bandage & Dressing Application

- Acute & Chronic Wounds
- Post-Surgical Applications
- Abrasions
- Burns

**HYPOCHLOROUS TECHNOLOGY**

**non-sensitizing**

**non-cytotoxic**

**non-irritating**

**Dispensed by licensed veterinarians only.**

**Indications:** Intended for use by veterinary professionals for moistening and debriding wounds of all types, burns, pressure sores, ulcers, abrasions, dermal and mucosal irritations, post-surgical incisions, grafted and donor sites, in addition to moistening and lubricating absorbent wound dressings. Vetericyn Plus<sup>®</sup> VF Antimicrobial Wound Barrier Gel relieves itch and pain often associated with dermal irritations. Promotes a moist environment which has been proven to help wounds heal faster.

**Directions for use: Shake well before each use.** Remove excess hair and necrotic tissue from wound area. Necrotic tissue can be softened with hydrogel for easier debridement. Twist open cap and saturate the affected area. Cover with bandage material if required. Hydrogel is designed to adhere to the site and evaporate slowly. The thicker formula acts as a protective barrier while maintaining moisture in the wound to help promote healing. For best results clean affected area with Vetericyn Plus<sup>®</sup> VF Wound & Skin Cleanser prior to applying hydrogel. For more accurate and precise application, draw up from cap with slip-on syringe.

**Use Precautions:** For external use only, not for injection. For animal use only, not for human use. Keep out of reach of children. Always consult with a veterinarian for systemic conditions. Discontinue use if irritation develops and contact your veterinarian.

**Storage:** Store at room temperature away from direct sunlight and heat. Do not allow to freeze. Do not use after expiration date. After use, seal or close dispenser.

**Stability:** See Expiration Date

**Disposal:** No special disposal requirements. Vetericyn Plus<sup>®</sup> VF is environmentally friendly.

**Ingredients: Active:** Hypochlorous Acid (0.012%) **Inactive:** Electrolyzed Water, Sodium Chloride, Sodium Hypochlorite, Lithium Magnesium Sodium Silicate, Sodium Bicarbonate, Phosphates, Sodium Sulfate

Innovacyn products are manufactured under strict quality guidelines of ISO 13485 and FDA QSR. Vetericyn Plus<sup>®</sup> hydrogel formula contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the solution.

Manufactured & Bottled by Innovacyn, Inc., 3546 N. Riverside Ave, Rialto CA 92377

**Made in the USA**

**vetericynVF.com**

**866.318.3116**

	Part Number:
3 fl oz (90 ml)	2047; 20047 v1.0
8 fl oz (237 ml)	2045; 20045 v1.0

**CPN:** 1433046.0

---

By using this content, you agree to the [Terms of Service](#) and [Privacy Policy](#).

Copyright 2020 - Animalytix LLC