

**INNOVACYN, INC.**  
**3546 N. RIVERSIDE AVE., RIALTO, CA, 92377**

Toll-Free: 866-318-3116  
Fax: 909-428-1947  
Website: [www.innovacyn.com](http://www.innovacyn.com)  
Website: [www.vetericyn.com](http://www.vetericyn.com)  
Email: [customerservice@innovacyn.com](mailto:customerservice@innovacyn.com)

---

Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

---

## **VETERICYN PLUS® VF ANTIMICROBIAL WOUND & SKIN CLEANSER**

### ***Innovacyn***

Wound Bed Preparation  
Cleanser & Irrigation

- Acute & Chronic Wounds
- Post-Surgical Applications
- Abrasions
- Irritations/Burns

### **HYPOCHLOROUS TECHNOLOGY**

**non-sensitizing**

**non-cytotoxic**

**non-irritating**

**Dispensed by licensed veterinarians only.**

**Indications:** Intended for use by veterinary professionals for cleansing, irrigating, moistening, and debriding wounds of all types, dermal and mucosal irritations, pressure sores, ulcers, abrasions and post-surgical incisions, grafted and donor sites.

**Directions for use:** Remove excess hair and necrotic tissue from wound area. Remove spray cap and saturate the affected area, flushing away any foreign debris (3 fl oz). Adjust spray nozzle to desired setting (stream setting is best for irrigating wounds) and saturate the affected area, flushing away any foreign debris (16.9 fl oz). Repeat as needed or as directed by veterinarian. No rinsing necessary. Faster healing is achieved when wounds are clean and maintained in a moist environment. Safe for use around eyes, ears, nose and mouth.

**Use Precautions:** For external use only, not for injection. For animal use only, not for human use. Keep out of reach of children. Always consult with a veterinarian for systemic conditions. Discontinue use if irritation develops and contact your veterinarian. Safe for use around eyes, ears, nose and mouth.

**Storage:** Store at room temperature away from direct sunlight and heat. Thaw and shake contents before use if frozen. Do not use after expiration date. After use, seal or close dispenser.

**Stability:** See Expiration Date

**Disposal:** No special disposal requirements. Vetericyn Plus® VF is environmentally friendly.

**Ingredients: Active:** Hypochlorous Acid (0.024%) **Inactive:** Electrolyzed Water, Sodium Chloride, Phosphates, Sodium Hypochlorite  
Innovacyn products are manufactured under strict quality guidelines of ISO 13485 and FDA QSR. Vetericyn Plus liquid formula contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the solution. Manufactured & Bottled by Innovacyn, Inc., 3546 N. Riverside Ave, Rialto CA 92377

**Made in the USA**

**[vetericynVF.com](http://vetericynVF.com)**

**866.318.3116**

	Part Number:
2 fl oz	2002; 90055 v3.2
3 fl oz (90 ml)	2004; 20004 v1.0
16.9 fl oz (500 ml)	2000; 90001 v5.0

**CPN:** 1433002.9

---

By using this content, you agree to the [Terms of Service](#) and [Privacy Policy](#).

Copyright 2020 - Animalytix LLC