

INNOVACYN, INC.
3546 N. RIVERSIDE AVE., RIALTO, CA, 92377

Toll-Free: 866-318-3116
Fax: 909-428-1947
Website: www.innovacyn.com
Website: www.vetericyn.com
Email: customerservice@innovacyn.com

Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

VETERICYN PLUS® HOT SPOT ANTIMICROBIAL GEL

Vetericyn

VETERINARIAN RECOMMENDED

proven itch relief for irritated skin as a result of biting | scratching | chewing | licking

- Gels on contact and provides protection
- Non-toxic, safe if licked or ingested
- Relieves symptoms of skin allergies
- No alcohol, steroids or antibiotics

Indications: Vetericyn Plus® Hot Spot Antimicrobial Gel is intended for the management of wounds including skin abrasions, lacerations, irritations, cuts, burns and intact skin; providing a proper moisture balance throughout the phases of wound healing. Relieves itch and pain associated with dermal irritation. Safe for all animal species.

Storage: Store at room temperature. Do not allow to freeze. Do not use after expiration date. After use, rinse nozzle with warm water to reduce chance of hydrogel drying at the tip. Replace cap.

Stability: See Expiration Date

Disposal: No special disposal requirements. Vetericyn Plus is environmentally friendly.

Directions for Use: To prime for initial application: remove cap, depress and hold sprayer, shake bottle, release sprayer. Move excess hair from wound area. Clip if necessary. Saturate the affected area. If dressing is required, saturate dressing with Vetericyn Plus® Wound & Skin Care Liquid or Antimicrobial Hydrogel at each dressing change. Repeat 3-4 times per day or as necessary. Faster healing is achieved when wounds are cleaned and maintained moist. No rinsing necessary. Safe for use around mouth, nose, ears, and eyes. For best results, first clean affected area with Vetericyn Plus® Wound & Skin Liquid to ensure deep dermal cleansing and proper wound bed preparation.

Use Precautions: For external use only, not for injection. Always consult with a veterinarian for systemic conditions. Discontinue use if irritation develops and contact your veterinarian. For animal use only, not for human use. Keep out of reach of children. Safe if licked by animal. Will not stain and contains no harsh chemicals or dyes.

Ingredients: Active: Hypochlorous Acid (0.010%) **Inactive:** Electrolyzed Water, Sodium Chloride, Sodium Hypochlorite, Lithium Magnesium Sodium Silicate, Sodium Bicarbonate, Phosphates, Sodium Sulfate

Innovacyn products are manufactured under strict quality guidelines of ISO 13485 and FDA QSR. Vetericyn Plus Hydrogel formula contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the Hydrogel.

Made in the USA

Manufactured & Bottled by Innovacyn, Inc., 3546 N. Riverside Ave, Rialto CA 92377

vetericyn.com

866.318.3116

Tell Us Your Story!

	Part Number:
3 fl oz (89 ml)	1048; 10048 v1.3

CPN: 1433035.3

By using this content, you agree to the [Terms of Service](#) and [Privacy Policy](#).

Copyright 2020 - Animalytix LLC